

Ruhof AquaBrush

Item No. 345AQB - AquaBrush and 6 Brushes
Item No. 345LBR - 25 Large Replacement Brushes
Item No. 345SBR - 25 Small Replacement Brushes

The Only Totally Submersible Rechargeable Cleaning Brush for Surgical Instruments

The Ruhof AquaBrush is a rechargeable, completely submersible surgical instrument cleaning brush, which eliminates manual scrubbing for easier, more efficient instrument reprocessing. The AquaBrush comes with both large and small brush heads to accommodate various instrument sizes.

Features

- Rechargeable - cordless
- Durable and lightweight
- Comfortable to use
- Push button operation
- Disposable cleaning brush heads in two sizes - large and small
- Includes two lithium batteries with charger



Ergonomically designed non-slip pistol grip with push button control fits comfortably in your hand



Large Brush - 60mm



Small Brush - 41mm



Includes 6 scrubbing brushes and Fast Charger.



continued

Ruhof AquaBrush

Directions for Use

1. Follow the requirements of the OSHA Bloodborne Pathogens standard for both proper work practice protocol and the use of appropriate personal protective equipment (gloves, face and eye protection, gowns).
2. Attach the appropriate disposable brush head to the hand held unit. Use the large brush (60 mm) to clean larger instruments such as those used in orthopedic surgery etc.; use the small brush (41 mm) for smaller, more delicate instrumentation.
3. Gripping the ergonomically designed unit push the on/ off button to begin cleaning either with the unit submersed (recommended) or over the sink after being saturated with your mixed chemistry. The unit will stay on until the button is pressed again to turn it off.
4. Since the AquaBrush includes two lithium batteries it is recommended that the battery not being used be kept charged so that the AquaBrush is always ready for use; utilize the enclosed fast charger, which works in half the time of traditional chargers.
5. Replace the disposable brush heads daily, or sooner if deemed necessary. The surface of the AquaBrush unit should be wiped down with a surface germicidal disinfectant daily, or sooner if needed.

Warning

The disposable brush head accessories of the AquaBrush should be discarded daily. Any institution, practitioner, or third party who reprocesses, refurbishes, remanufactures, sterilizes, and /or reuses this disposable device must bear full responsibility for its safety and effectiveness.

Product Disposal

After daily use, the disposable brush heads may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.



393 Sagamore Avenue, Mineola, New York 11501
1-800-537-8463 • www.ruhof.com

